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KEDLEY™ ORTHOPAEDIC SUPPORTS

ENG FABRIC ARM SLING

Please read the following instructions and keep for future reference.

KEDLEY brings you a complete range of quality athletic supports and braces. **Kedley Orthopaedic** range is made from only the highest quality materials. Each item is manufactured to give you steady support to promote healing and comfort.

The Kedley ORTHOPAEDIC supports range is suitable for both sporting and everyday use.

Note: The support should be sized correctly to ensure maximum benefit and to avoid over compression. If the support is being used following an injury, seek medical advice if no improvement in the condition is seen within 72 hours. If the injury is severe or if in any doubt seek medical advice immediately. If used by children, adult supervision is required. Keep out of reach of young children.

Warning: If soreness or a rash occurs discontinue use immediately and seek medical advice.

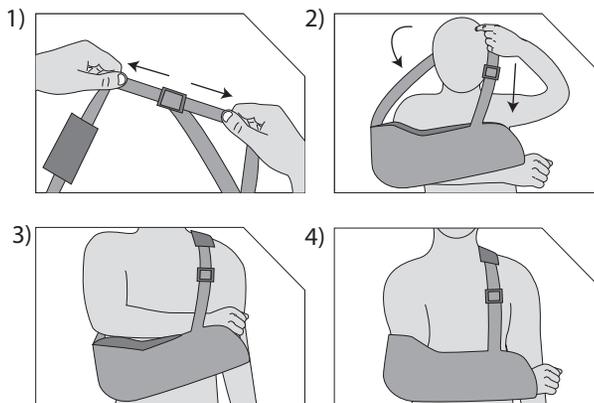
Washing Instructions: Hand wash in luke warm soapy water and air dry.

ORTHOPAEDIC SUPPORTS FABRIC ARM SLING

Product code: KED043/047

Website: www.kedley.co.uk

Address: Braitrim House,
98 Victoria Road,
London, UK,
NW10 6NB.

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