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## ENG ELBOW SUPPORT

Please read the following instructions and keep for future reference.

**KEDLEY** brings you a complete range of quality athletic supports and braces. The **PRO-LIGHT** range is made from neoprene rubber and infused with polyester as well as polyamide for extra strength, flexibility and comfort, to ensure a secure fit. The neoprene fabric is designed to retain body heat; this warming effect on the muscles and joints increases flexibility and helps prevent injuries. Each product is designed to give maximum support whilst maintaining a comfortable fit. Refer to insert for additional information inside the box.

The Kedley PRO-LIGHT Neoprene support range is suitable for both sporting and everyday use.

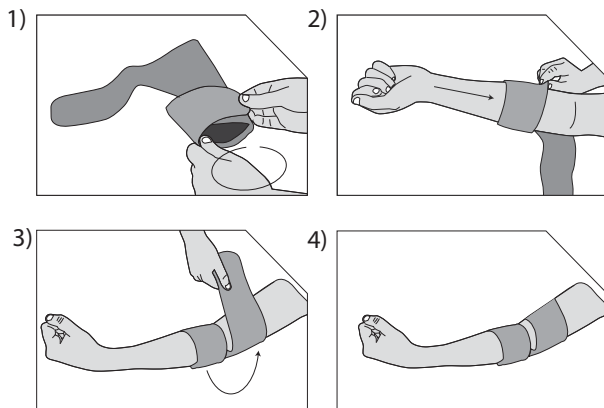
**Note:** The support should be sized correctly to ensure maximum benefit and to avoid over compression. If the support is being used following an injury, seek medical advice if no improvement in the condition is seen within 72 hours. If the injury is severe or if in any doubt seek medical advice immediately. If used by children, adult supervision is required. Keep out of reach of young children.

**Warning:** Do not wear neoprene supports if warming or heat generating liniments, ointments or creams have been applied. If soreness or a rash occurs discontinue use immediately and seek medical advice.

**Washing Instructions:** Hand wash in luke warm soapy water and air dry.

## PRO-LIGHT NEOPRENE RANGE

**Website:** [www.kedley.co.uk](http://www.kedley.co.uk)  
**Tel no.** +44 (0) 2037 326 516  
**Address:** Braitrim House,  
98 Victoria Road,  
London, UK,  
NW10 6NB.

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